

Three Year 100% Performance Guaranty / Warranty

Perspectis, Inc. ("Perspectis") warrants, subject to the conditions set forth below, that should this product be defective due to poor workmanship or materials any time during the specified warranty period, Perspectis will repair or replace the same with the latest model, whatever is fair, without charge for either parts or labour. Shipping charges may apply.

The warranty period is THREE years from the date of original purchase at retail.

CONDITIONS

1. **Registration:** The attached Warranty Registration Form must be submitted to Perspectis within forty five (45) days after the date of original purchase.
2. **Proof of Date of Purchase:** This warranty applies to the product from the original date of purchase at retail. Therefore, the owner must furnish proof of original purchase, should the Warranty Registration Form not be returned.
3. **Unauthorized repair, abuse, etc.:** The unit must not have been altered, modified, or repaired. The unit must not have been subject to accident, misuse, abuse, commercial use, rental, or operated contrary to the User Instructions.
4. This warranty does not extend to normal wear and tear and damage caused by misuse or improper care, does not cover regular product maintenance such as cleaning or lubrication, and is valid only within the country where the unit is originally purchased.
5. **Underwater bubble test** (submerge an inflated Dr. H. H. Belt in a water-filled sink and press on it) need to be performed by the owner before a warranty service inquiry is submitted.
6. **Proper delivery:** The owner must call or email Perspectis for authorization and instructions prior to returning the defective product for warranty inspection. Any unauthorized returns will be refused or shipped back at the owner's expenses. Upon receiving Perspectis' warranty authorization and instructions, the defective unit must be shipped, freight prepaid, to the designated Perspectis facility in either its original package or similar package affording an equal level of protection. The owner must include a note with their name, address and telephone number along with a description of the defect.

EXCEPT TO THE EXTENT PROHIBITED BY APPLICABLE LAW, PERSPECTIS, INC. EXPRESSLY DISCLAIM ALL OTHER WARRANTIES AND CONDITIONS, EXPRESS OR LIMITED, AND WHETHER ARISING BY LAW, BY STATUTE, BY COURSE OF DEALING OR USAGE OF TRADE, INCLUDING WITHOUT LIMITATION IMPLIED WARRANTIES OR CONDITIONS OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. UNDER NO CIRCUMSTANCE SHALL PERSPECTIS, INC. OR ITS SUPPLIERS, VENDORS OR DISTRIBUTORS BE LIABLE FOR AN AMOUNT GREATER THAN THE ACTUAL PURCHASE PRICE OF THE UNIT OR FOR ANY DIRECT, INDIRECT, SPECIAL, PUNITIVE, INCIDENTAL, EXEMPLARY OR CONSEQUENTIAL DAMAGES, OR ANY DAMAGE WHATSOEVER, RESULTING FROM ANY USE OF THIS PRODUCT, OR ANY OTHER CAUSES, WHETHER BASED ON PRODUCT LIABILITY, OR OTHERWISE, ARISING OUT OF OR IN ANY WAY CONNECTED WITH THE USE OR PERFORMANCE OF THIS PRODUCT, WITH THE INABILITY TO USE THIS PRODUCT, OR WITH THE PROVISION OF OR FAILURE TO MAKE AVAILABLE ANY OF ITS OR THEIR PRODUCTS, GOODS, OR SERVICES, EVEN IF ADVISED OF THE POSSIBILITY OF SUCH DAMAGES. AND PERSPECTIS, INC. NEITHER ASSUMES NOR AUTHORIZES ANY REPRESENTATIVE OR OTHER PERSON TO ASSUME FOR IT AN OBLIGATION OR LIABILITY OTHER THAN AS IS EXPRESSLY SET FORTH HEREIN. THIS PRODUCT IS OFFERED TO YOU CONDITIONAL UPON YOUR ACCEPTANCE WITHOUT MODIFICATION OF THE LIMITATIONS SET FORTH ABOVE. TEST AND/OR USE OF THIS PRODUCT IMPLIES ACCEPTANCE OF ALL SUCH LIMITATIONS.



DR. HAMILTON HALL BELT™

User Instructions & Three Year 100% Performance Guaranty / Warranty (must read first)



Customers located in USA and Canada:
Please **DO NOT** contact place of purchase for warranty assistance.
Call toll free 1-855-BodyRyzm for any warranty questions or services.

BodyRyzm LifeScience / Perspectis, Inc.
1 First Canadian Place, Suite 350
Toronto, ON, Canada, M5X 1C1
www.BodyRyzm.com
Toll free: 1-855-BodyRyzm (263-9799), Tel: 1-416-595-1575
Fax: 1-416-595-6438
Email: CustomerService@BodyRyzm.com



WARNING



- This product is not a toy. All users need to read and understand all instructions in this document before using or testing this product.
- Start with low inflation – no more than 1/2" in total thickness of an inflated Dr. Hamilton Hall Belt, especially when you just begin to use your Dr. Hamilton Hall Belt. Less is more! You may increase the level of inflation depending on your needs and preferences after several nights of usage.
- Do not jump on this product.
- Do not sit or step on this product.
- Do not use this product or do any exercise on or with this product, if you have acute back pain, injury, recent surgery, pregnancy or any other unusual condition. Consult your licensed and qualified health advisor (hereinafter "Advisor") if you have any questions.
- Use this product across the small of your back only, when lying on it.
- This product is designed for weight loads up to 350 lbs. only.
- Keep this product away from any sharp objects, any aggressive or caustic chemicals, and any intense heat source.
- Adjust the inflation of this product, only when it is at room temperature.
- Please keep the packaging bag away from babies and children to avoid danger of suffocation.

HOW TO ADJUST, CARE FOR AND TROUBLESHOOT...

How to adjust the level of support provided by Dr. Hamilton Hall Belt

You can do so by easily and instantly adjusting the inflation of Dr. Hamilton Hall Belt:

- Open the valve. Gently inflate by blowing with mouth or deflate by pressing on Dr. Hamilton Hall Belt and let air in or out to achieve your desired level of inflation. Please start with low inflation when you begin to use your Dr. Hamilton Hall Belt. Less is more!
- Close the valve firmly and enjoy your Dr. Hamilton Hall Belt. This adjustment can be done within 30 seconds.

How to care for your Dr. Hamilton Hall Belt

- Place a hand towel over your Dr. Hamilton Hall Belt when lying on it.
- Re-adjust the inflation of the device once every week if needed.
- Clean your Dr. Hamilton Hall Belt by wiping it with soft cloth and warm soapy water whenever necessary.

HOW TO ENJOY...

Reduce back pain by providing the required resistance and support under lumbar spine while sleeping

Support Your Lower Back



- Inflate your Dr. Hamilton Hall Belt slightly and close valve firmly. Please start with low inflation. You may increase inflation depending on your needs and preferences after several nights of usage. Please **note** that the optimum level of support for a good sleep overnight is usually less than the level of support often preferred for a short period of rest.
- Place it flat on your bed.
- Lie gently on your Dr. Hamilton Hall Belt. Make sure that your Dr. Hamilton Hall Belt is across the small of your lower back, and perpendicular to your body.
- There is no need to do anything special. Just lie on it. And your Dr. Hamilton Hall Belt will do the rest.

Reduce back pain by decompressing lumbar spine

Decompress Your Lower Back



- Inflate your Dr. Hamilton Hall Belt and close valve firmly. You can increase inflation if you prefer stronger support after several times of usage.
- Place it flat on a flat surface such as bed, floor, sofa, etc.
- Lie gently on your Dr. Hamilton Hall Belt. Make sure that your Dr. Hamilton Hall Belt is across the small of your lower back, and perpendicular to your body.
- There is no need to do anything special. Just lie on it. And your Dr. Hamilton Hall Belt will do the rest.
- Lie on your Dr. Hamilton Hall Belt for 20 minutes at a time as needed.